



TITAN CSR® RETRACTOR 4-Ez EXPOSURE METHOD™





1ENGAGE TITAN CSR®
retractor blades with abdominal wall

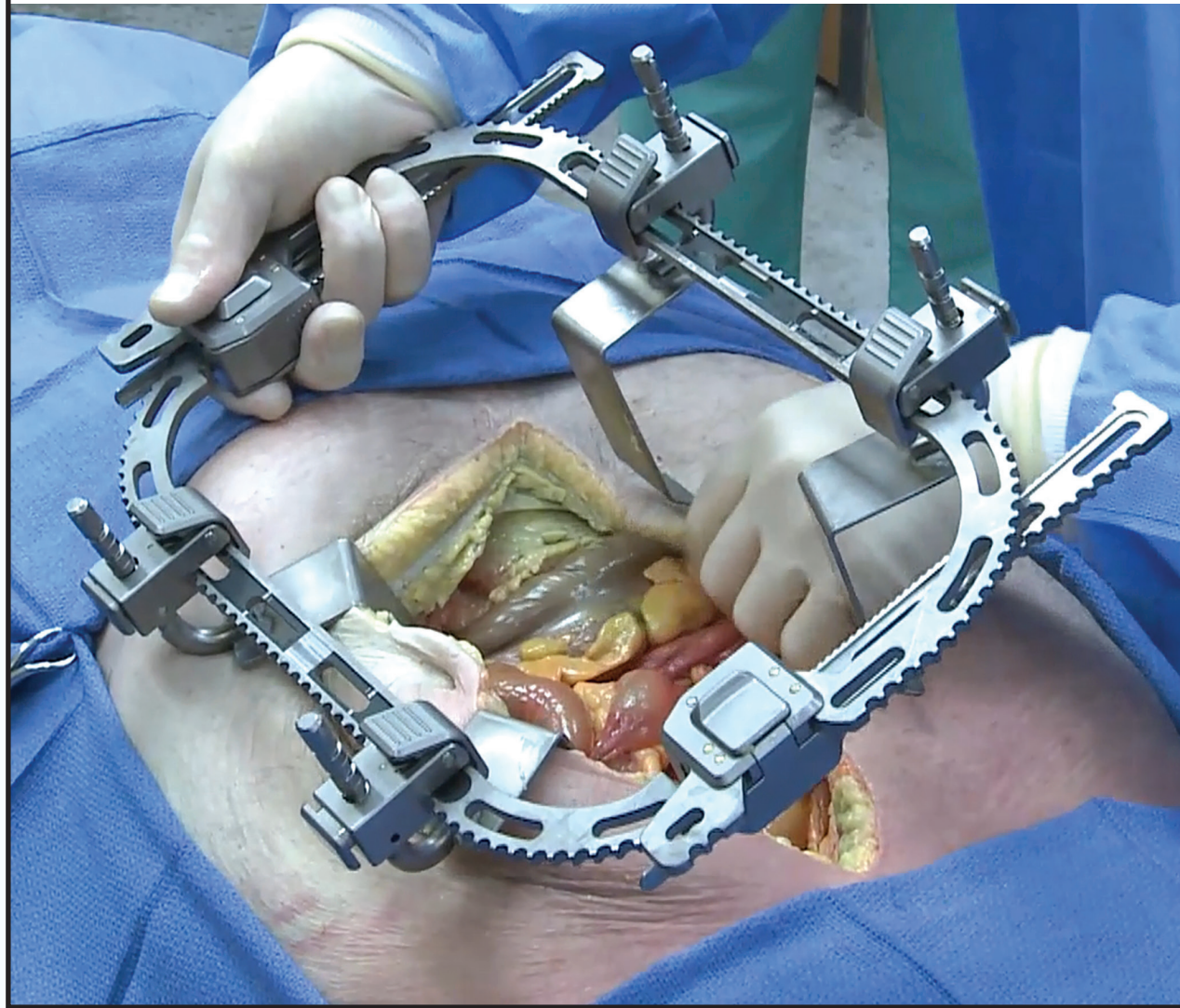


2EXPAND with moderate force
to generate tension on abdominal wall

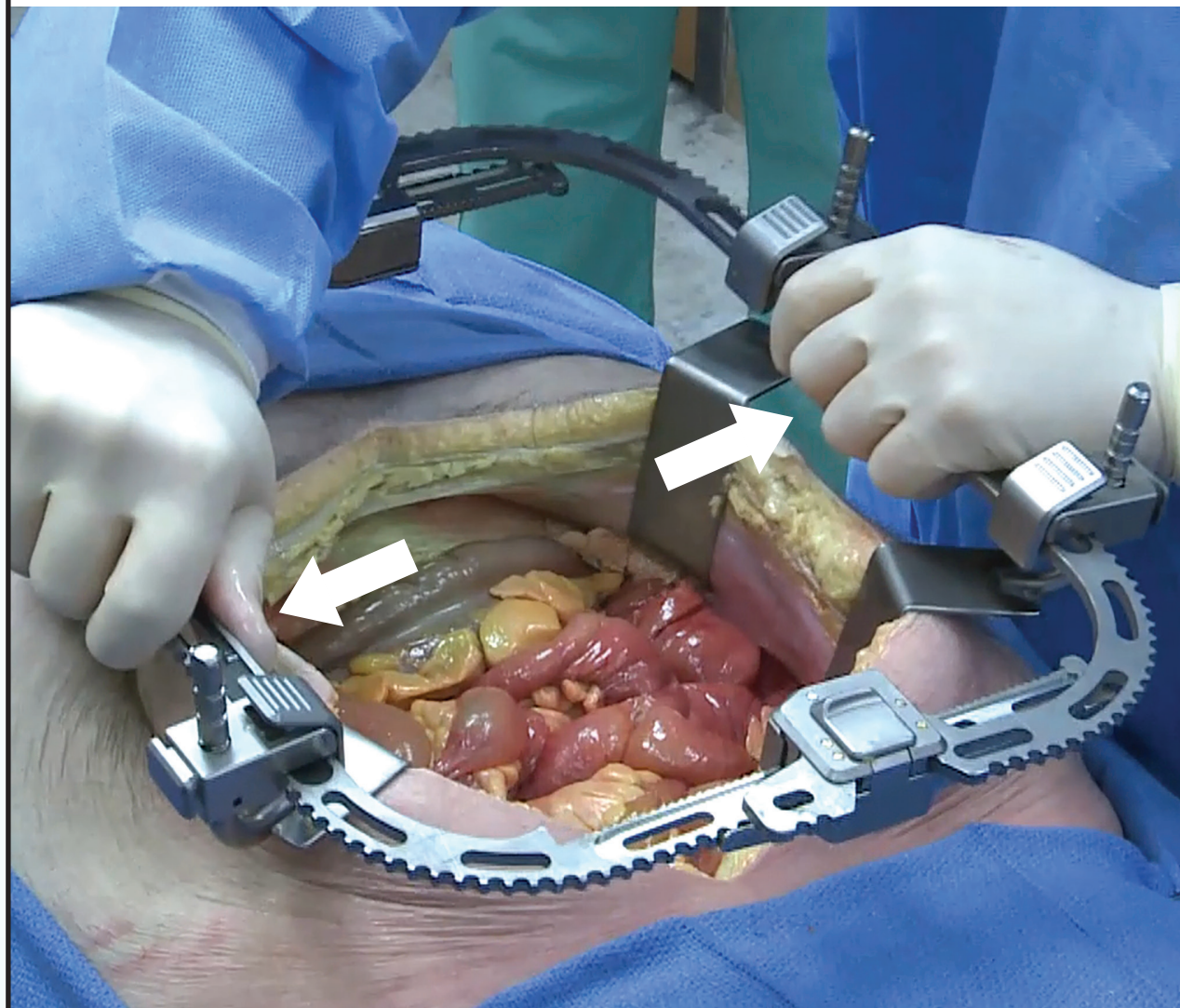


3EXPOSE abdominal wall with 4 Bookwalter®
blades at corners; malleables for retroperitoneum

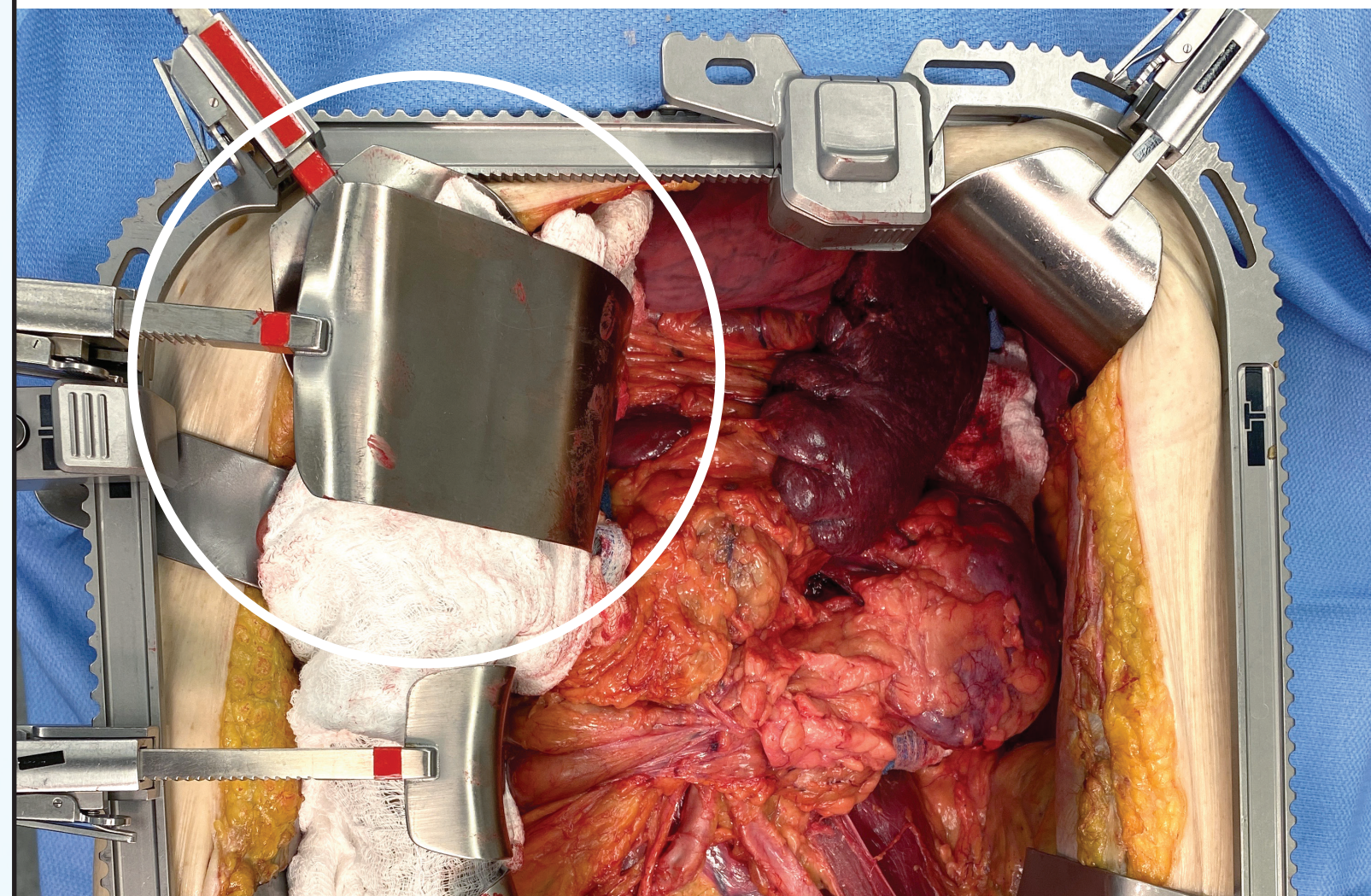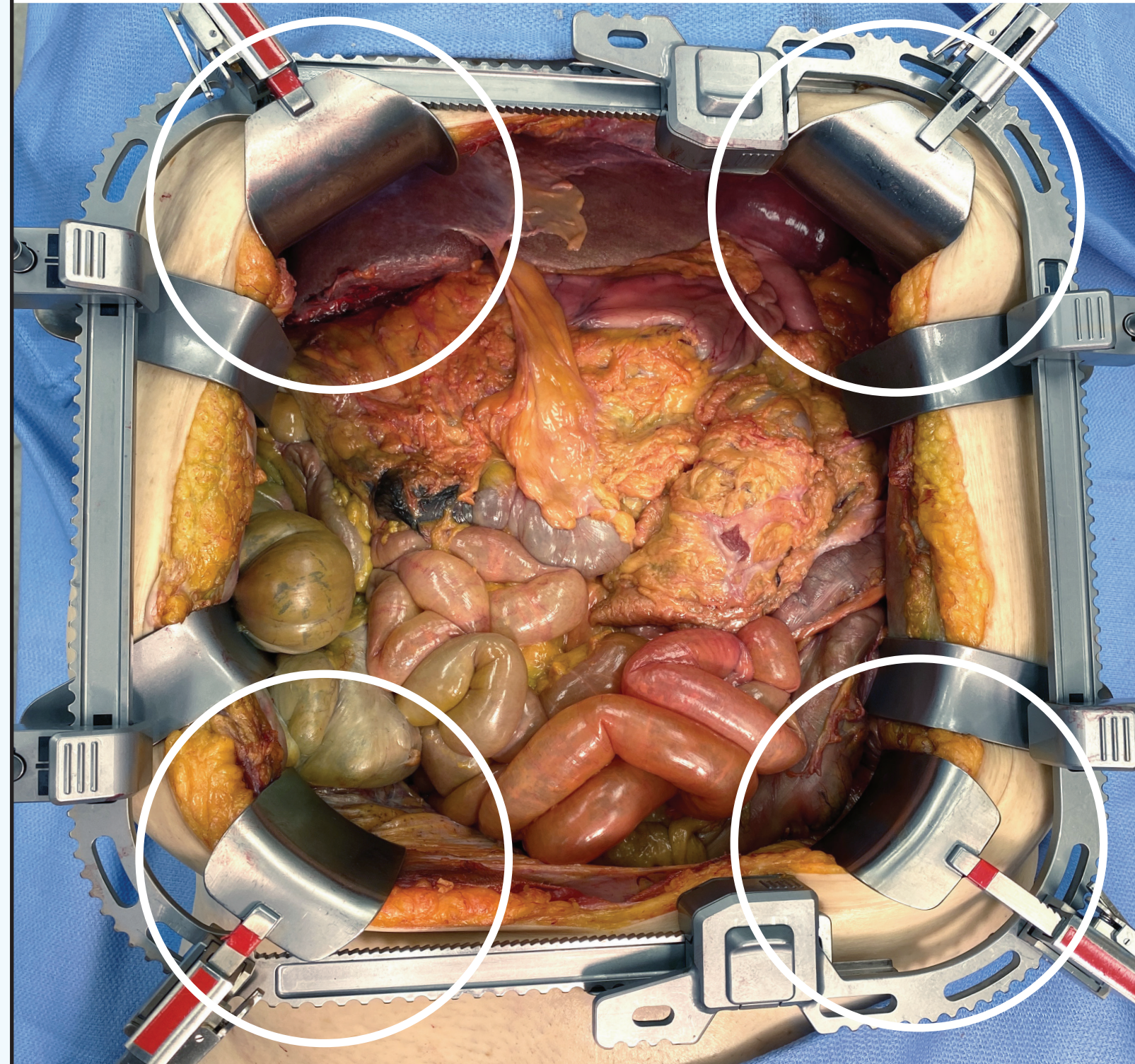

4ELEVATE TITAN CSR®
retractor frame or Bookwalter® blade
handles for abdominal packing and full
exploration

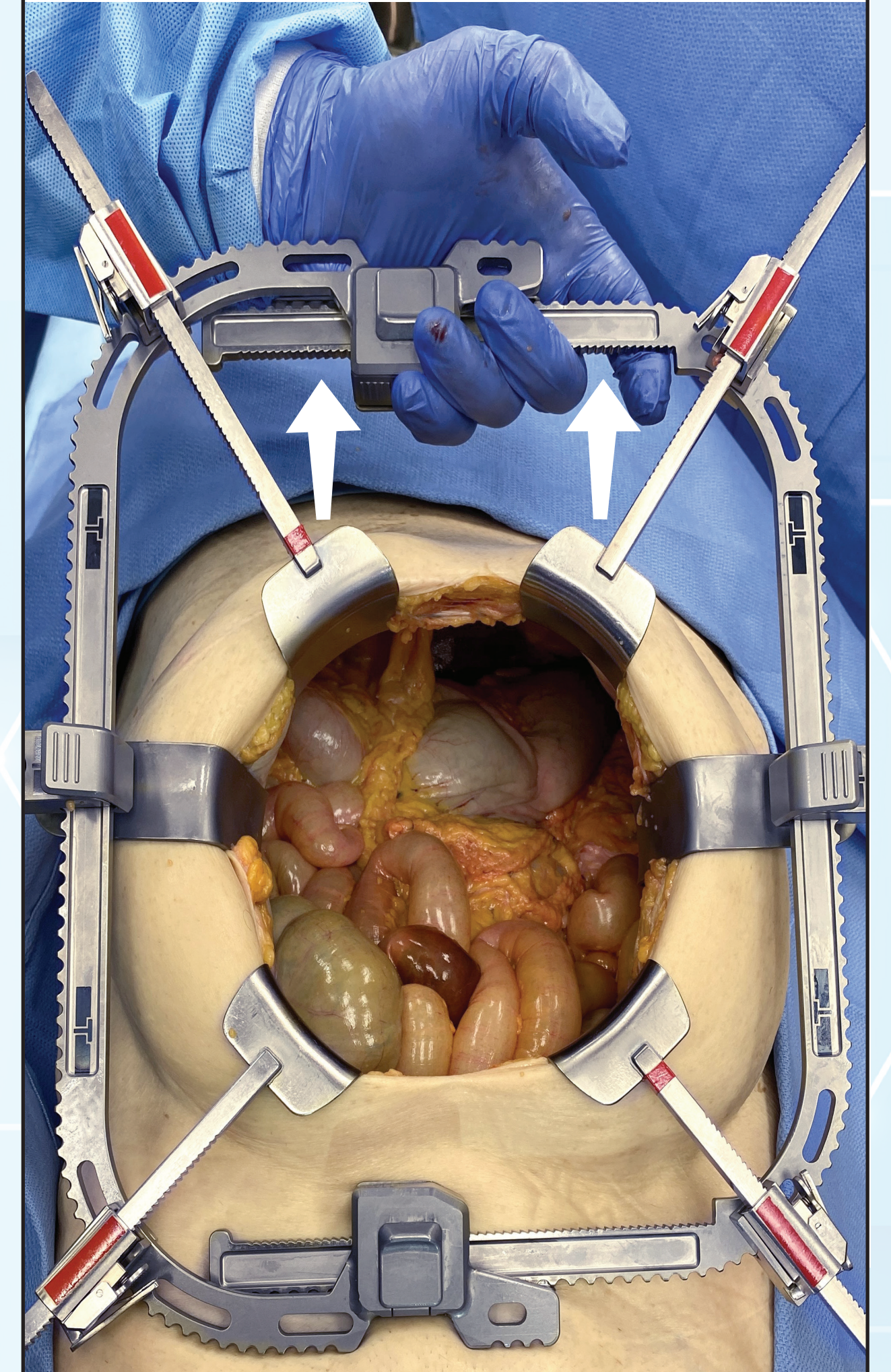

*The Bookwalter® trademark is a registered trademark of Symmetry Surgical, Inc., and ASR Systems makes no claim to it. ASR Systems is not an authorized seller of Bookwalter® branded products.

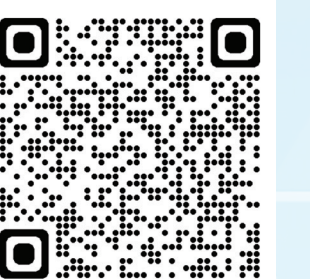

[titanretractor.com](https://www.titanretractor.com)